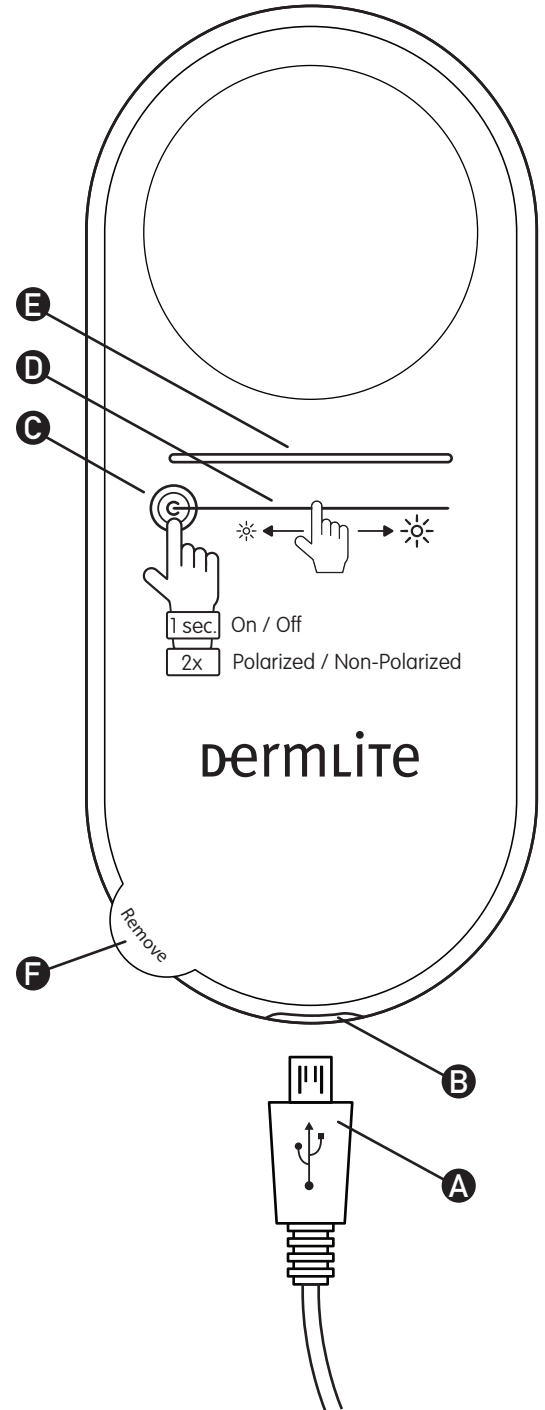
# dermLite<sup>®</sup> Lumio<sup>®</sup>S

## Instructions DermLite Lumio S™ Skin Examination Device

**CAUTION:** Prior to using Lumio S, read these instructions and keep them in a safe place for future reference.  
**CAUTION:** United States Federal Law restricts this device to sale by or on the order of a physician.  
**CAUTION:** The DermLite cam is designed for external examination only.

Lumio S is a diagnostic instrument designed for cross-polarized and non-polarized examination of the skin. In cross-polarized mode, glare-free visualization is accomplished by canceling the reflected light from the skin.

Directions for Use  
**CAUTION:** Do not look directly into the bright LED light. Patients must close their eyes during facial examinations.

**IMPORTANT:** Before initial use, please charge your Lumio S for a minimum of 2 hours using the included USB cable.

**Charging**  
Prior to using Lumio S, plug the smaller, Micro USB end (A) of the USB cable into the device's USB jack (B), and the larger end into an available USB port (e.g. on your desktop PC, laptop or phone charger). The Indicator Strip (E) scrolls blue while charging and shows solid blue once Lumio S is fully charged.  
In lieu of charging via USB cable, the 'Lumio S Charging Pad' (sold separately) may be used. Above behavior of the Indicator Strip is unchanged.

After about 1,000 charges, the battery's capacity will diminish and the battery may need to be replaced. Contact 3Gen for more information.

**Using Lumio S**  
The device is controlled with one finger via a two part touch-sensitive region, comprising a C-shaped Control Area (C) on the very left and a Slider Bar (D). To turn on the device, tap & hold the Control Area for 1 full second. To change between cross-polarized and non-polarized modes, double-tap the C. When in cross-polarized mode, the Indicator Strip (E) illuminates red; green indicates non-polarized lighting. To change LED brightness, swipe along the Slider Bar from left to right (or from right to left), while the Indicator Strip signifies current brightness at all times. Tap & hold the C to turn off the device. To preserve battery life, the device will turn off automatically after 3 minutes of inactivity.

Once you are familiar with how to control the device, peel off the instructional label (F).

**CAUTION:** Your Lumio S uses a unique lithium-polymer battery, which can only be purchased from 3Gen or from an authorized 3Gen dealer. Do not under ANY circumstances use another battery other than the one

designed for this unit. Doing so could result in damage to the unit.

### Troubleshooting

Please check our website ([www.derm-lite.com](http://www.derm-lite.com)) for the most current troubleshooting information. If your DermLite requires servicing, please go to the "Service & Support" pages at [www.derm-lite.com](http://www.derm-lite.com), where you can complete a "Repair Request" form. Or, contact your local 3Gen dealer.

**No Light**  
Fully recharge the battery and check again. If the condition persists, your DermLite may require servicing.

**LED Fails**  
The LEDs used in your DermLite are designed to last over 100,000 hours. If any one of the LEDs fails, your DermLite requires servicing.

**Mechanical Malfunction**  
Please contact 3Gen to arrange for service.

### Care and Maintenance

Your DermLite is designed for trouble-free operation. There are no adjustments to be made nor is any special servicing required. Unless instructed to do so by an authorized 3Gen representative, NEVER attempt to open the device for any reason.

**Cleaning and Disinfection**  
The exterior of your DermLite may be cleaned or disinfected by wiping it with Isopropyl alcohol (70% vol.). Do not use abrasive material on any part of the equipment or immerse the device in liquid. Do not autoclave.

**Cleaning the Glass**  
The glass top and bottom surfaces of the device should be treated as high quality photographic equipment and should be cleaned with standard lens cleaning equipment and protected from harmful chemicals.

Lumio S components include:  
Lumio S, USB cable.

**Warranty**  
5 years for parts and labor.

Serial number and company information are located on the bottom of the Lumio S.

**REF** Model Number  
**SN** Serial Number  
 Attention, consult accompanying document  
 Manufacturer  
 Do not dispose as unsorted municipal waste  
 EC Representative (EU Only)

M. Devices Group, Marlborough House,  
Riding Street, Southport, PR8 1EW, UK.  
Tel: +44 1704 544 944 Fax: +44 1704 544 050  
 M. Devices Group, Marlborough House,  
Riding Street, Southport, PR8 1EW, UK.  
Tel: +44 1704 544 944 Fax: +44 1704 544 050

Patents: [www.derm-lite.com/patents](http://www.derm-lite.com/patents)

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