

Recommendations for disinfectant wipes

MAINTENANCE OF THE VIVATMO SYSTEM

The following disinfectant wipes are recommended for cleaning and disinfection purposes of the Vivatmo *pro* and the Vivatmo *me*:

Manufacturer	Disinfectant wipe
Bode Chemie GmbH	Mikrobac® Tissues
Schülke & Mayr GmbH	mikrozid® universal wipes
Reckitt Benckiser	Sagrotan wipes
Metrex	CaviWipes1™
Dr. Schumacher GmbH	CLEANISEPT®
B. Braun Deutschland GmbH & Co. KG	Meliseptol® Wipes ultra
cosiMed GmbH	cosiMed Desinfektionstücher
Alfred Becht GmbH	Bechtifix alcohol free; Becht XL Wipes alcohol free

The use of any other unsuitable disinfectant wipes not listed above may cause malfunction or damage to the Vivatmo *pro* and the Vivatmo *me*.

Please note that the instructions for use of the Vivatmo *pro* and the Vivatmo *me* must be followed at all times.

CLEANING AND DISINFECTING

Clean the device regularly or when used by another patient as well as in accordance with the hygienic standards of your organization.

1. Turn off the device. Wash hands thoroughly with soap and water.
2. Wipe the device with disinfectant wipes with maximum 30 % alcohol. Examples are stated above.
3. For disinfecting repeat step 2 with disinfectant wipes with maximum 30 % alcohol following the wipe's manufacturer's instructions for disinfecting.

For further questions please contact your BHCS key account manager or the Vivatmo Customer Support under support@vivatmo.com.



Order here!



PRAXISDIENST

Medical Supplies since 1953

Sender / imprint

Bosch Healthcare Solutions GmbH
 Stuttgarter Str. 130
 71332 Waiblingen, Germany
 E-Mail: support@vivatmo.com
 Phone: +49 711 811-58313