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**PAX Rettungs-BOA / PAX Rescue-BOA**

Order here!

PRAXISDIENST  
Medical Supplies since 1953

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### Instructions for use PAX Rescue-BOA

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### PAX Rescue-BOA for the professional application!

Quick evacuation of the patient by means of the PAX Rescue-BOA tested according to DIN EN 1865-1. Delivery is carried out in a practical protection bag. In case of emergency the incorporated fleece blanket can be used for conservation of body heat.

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Outside: PES Polyester FR with dirt-repellant polymer coating

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## Material: PAX Rescue-BOA

*The PAX Rescue-BOA consists of a high-viz yellow coating out of 100% polyester with a waterproof polyurethan coating, a rolled up antibacterial fleece blanket and a special coated belt strap for the hand loops.*

- Coating:** *The coating out of polyester is bacteria and virus proof, flame retardant according to DIN EN ISO 12952 -1 & -2, suitable for chemical washing, dryer proof, steam sterilisable and suitable for dry cleaning and resistant against blood, urine, oils and fats as well as against surface disinfectants.*
- Fleece blanket:** *The fleece blanket consists of 100% polyester with antibacterial properties.*
- Lifting sling with carrying sling:** *The hand loops are manufactured from a coated belt strap, are easy to clean and also suitable for wipe disinfection with all common surface disinfectants.*

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## Intended purpose

*The PAX Rescue-BOA is used in order to evacuate a patient / person from an environment that makes a purposeful work impossible or when time pressure demands it. Moreover it serves to optimally immobilise the cervical and thoracic spine during and by proper use under the present circumstances. The application is restricted to the pre-hospital care and to the first phase of clinical care. Open wounds, fractures and injuries have to be treated according to the current rules of the specialty societies prior to the application. This product is about a medical product which is only to be used by briefed and regularly product trained skilled-personnel. The type of application follows the current, generally recognized medical directives and recommendations of the corresponding specialty societies. Furthermore the incorporated fleece blanket can be used as emergency warmth blanket.*

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## Reusability:

*The product can be reused when functionality is maintained and when the material is undamaged. Prior to a reapplication the user has to check and ensure the proper condition of the product. Only products of proper technical and hygienic condition may be reused.*

**The product has a lifetime of max. 5 years from purchase date.** Every year of use the person in charge of medical products has to carry out a visual inspection of all seams, connections, handles and surface material and to record it at the end of these instructions for use. Defective products may not be used anymore.

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## Combination of medical devices

*Basically the PAX Rescue-BOA may only be used in combination with a cervical collar correctly adjusted to its size (see series of pictures). When technically correctly used the PAX Rescue-BOA allows an optimal immobilization of the patient's spine under the present circumstances.*

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## Indication:

*The PAX Rescue-BOA can be used in order to evacuate a patient with a body height of 150 cm and more from his environment. The rescue sling serves as immobilization of the cervical and thoracic spine during the evacuation of the patient. Another field of application is the conservation of body heat of the patient. Once taken the PAX Rescue-BOA out of its cover it can be unrolled after opening the velcro fastener. After this procedure a square blanket surface with an edge length of 2,15 m appears.*

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## Contraindication:

The PAX Rescue-BOA may not be used when by means of its use the patient could be harmed more than acceptable under the circumstances. This e.g. counts for foreign objects located in those areas of the body that would get in direct contact with the PAX Rescue-BOA.

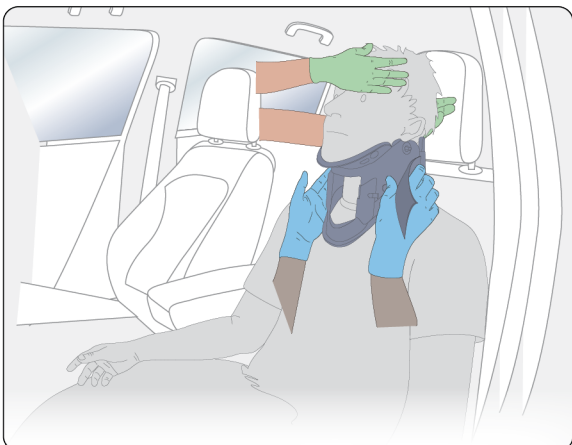
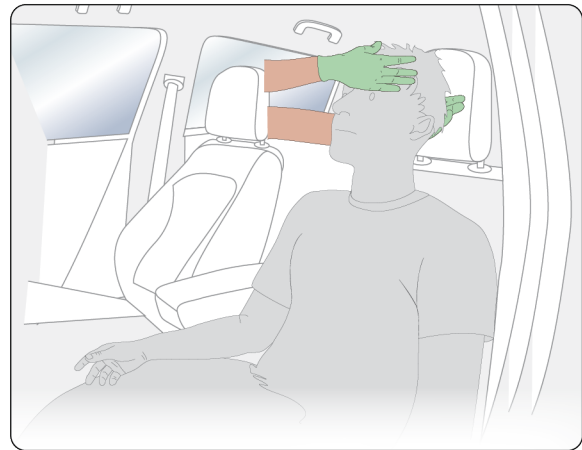
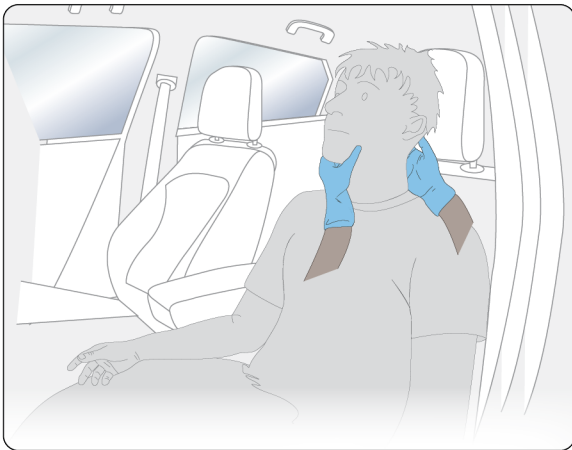
**Farbcodierung Zeichnung / Colour coding:**  zu rettende Person / Person to be rescued  Helfer 1 / Helper 1  
 Helfer 2 / Helper 2  Cervicalstütze / Cervical collar  PAX Rettungs-BOA / PAX Rescue-BOA

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## Preparation and application of the BOA's use:


In order to immobilize the cervical and thoracic spine right from the beginning, it is indicated to have the patient's head secured by helper 1. Then helper 2 can take over this immobilization from the corresponding alternative vehicle side as well as from the back seats.

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Thus then helper 1 can fix a cervical collar to the patient. In doing so attention is to be paid to the right size and to the correct fixing.

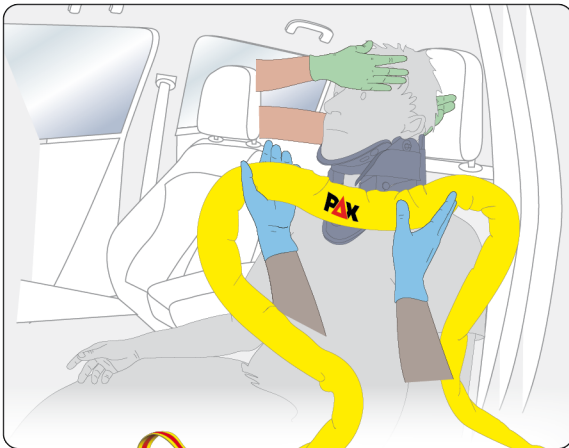Farbcodierung Zeichnung / Colour coding:  zu rettende Person / Person to be rescued  Helfer 1 / Helper 1  
 Helfer 2 / Helper 2  Cervicalstütze / Cervical collar  PAX Rettungs-BOA / PAX Rescue-BOA

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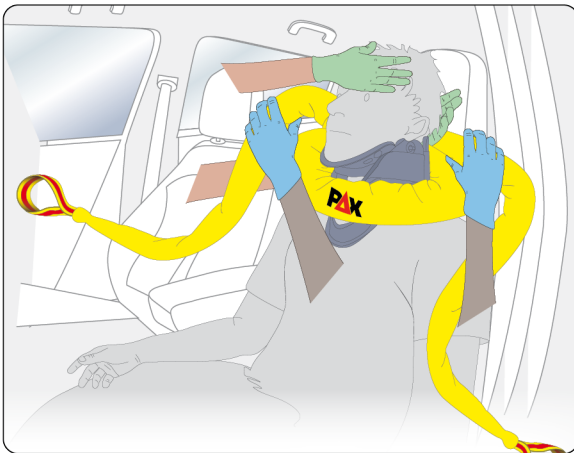
*Now the centre mark (tip of the red triangle in the PAX logo) is positioned in front of the larynx of the patient by helper 1 and both ends of the PAX Rescue-BOA are lead over the shoulders to the back.*

**Also important at this stage is the immobilization of the head to be ensured by helper 2.**

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*Both ends of the PAX Rescue-BOA are now crossed at the dorsal area of the cervical spine by helper 1. Now helper 1 pulls open the PAX Rescue-BOA in order to ensure immobilization of the cervical spine.*

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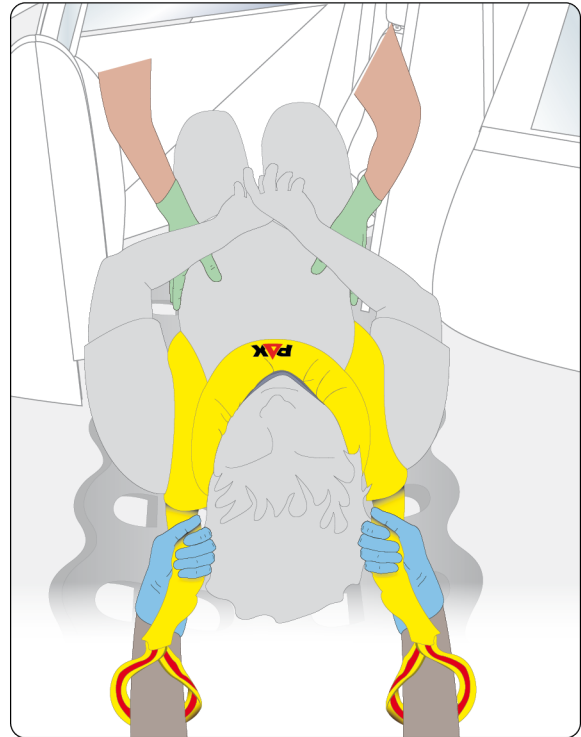
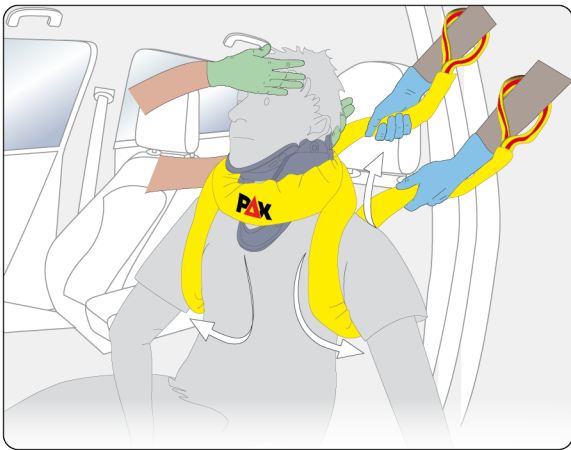
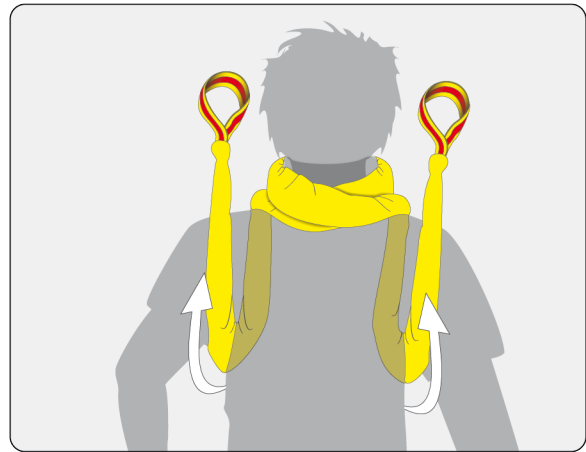
*Then the ends of the PAX Rescue-BOA are brought to the ventral area and are lead under the particular armpit to the back.*

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*Now helper 1 prepares the spine board (or other comparable rescue systems). It must be secured from sliding away and has to be positioned slightly below the upper edge of the seat.*

*Especially with comatose patients it is recommended to use the PAX-immobilization-hand loop (see following pages). It hinders the arms drifting apart during further rescue measures.*

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*Helper 1 now reaches for the PAX Rescue-BOA through both safety slings so that the hands remain above the shoulders.*

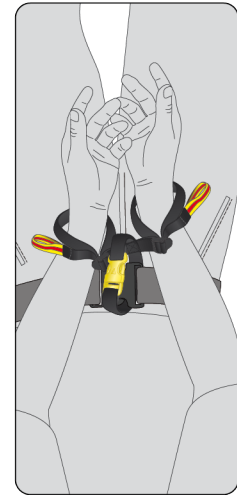
*The tractive forces to generate now to the PAX Rescue-BOA immobilize the cervical spine of the patient in such a way that manual immobilization of his head can be stopped by helper 2.*

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*Now helper 1 pulls the patient onto the prepared rescue device; doing that helper 2 supports the rotation of the body. In order to lay down the patient axis-aligned onto the rescue device helper 2 controls and corrects the body position of the patient during the pulling process. Since a hyperflexion of the cervical spine could harm the patient attention is to be paid to the patient's head being able to freely slide towards the cranial without being lifted.*

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*When the patient is properly secured to the rescue device the PAX Rescue-BOA can be loosened. The untying process is especially easy and quickly to do thanks to the shape getting thinner and thinner coming to the ends. Fixation of the wrists from the patient by means of the PAX-Immobilization-hand loop guarantees a constant positioning of the arms.*

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### PAX Immobilization-Hand loop

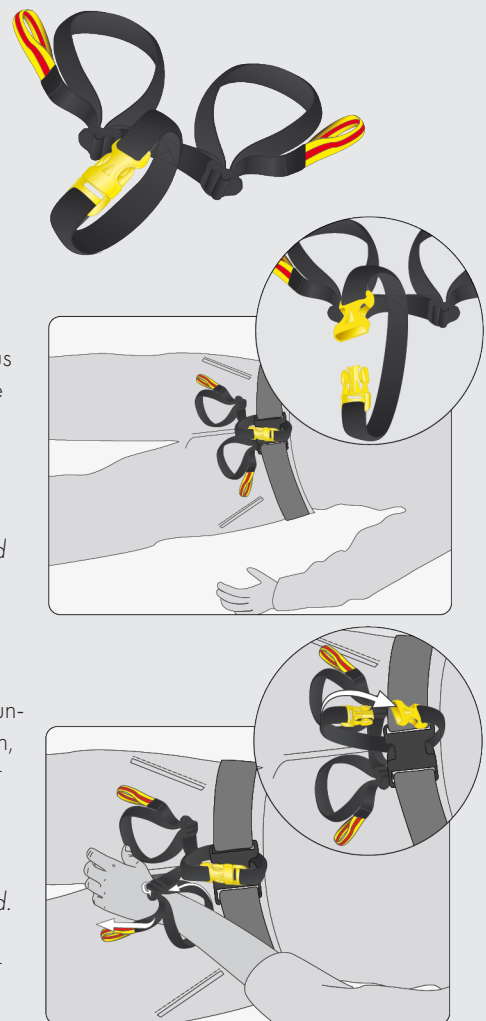
*In addition to the PAX Rescue-BOA (but also applicable with other rescue devices) the PAX Immobilization-hand loops are available. Especially with comatose patients it is necessary to position the arms tight to the body. In order to avoid arms hanging down in an uncontrolled way these can be easily fixed with the PAX Immobilization-hand loop.*

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*When locked the PAX Immobilization-hand loop consists of three single loops. Using the loop with the yellow buckle, the PAX Immobilization-Hand loop can be fixed to the patient's trousers or belt.*

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*When the loop is fixed the hand loops are positioned below the waistband. Now the hands can be fixed to the hand loops. To do that the loops can be wide opened by means of the belt loops and can be tightened up after positioning of the hand (wrists) in order to avoid the hands slipping out.*

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*In that way the arms can be positioned quickly and safely one after another and the evacuation of the patient can be continued. The belt loops can be easily loosened when the loops are not needed anymore. It is also possible to open the loops.*

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### Protection and warmth blanket within the PAX Rescue-BOA

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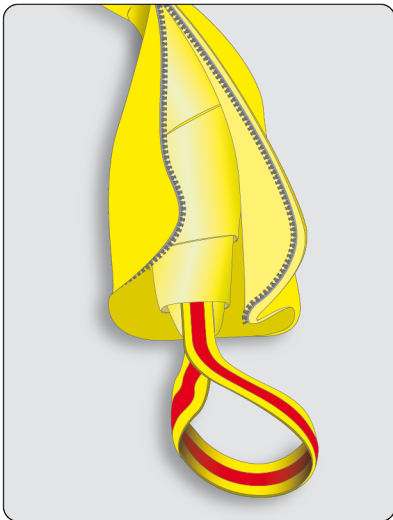
▲ By removing the cover the incorporated fleece blanket can be used as emergency warmth blanket after unrolling it.

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▲ After washing (see washing instruction) the warmth blanket can be packed into the cover again.

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
▲ In the ready-to-use condition the PAX Rescue-BOA is stored in the transport bag.

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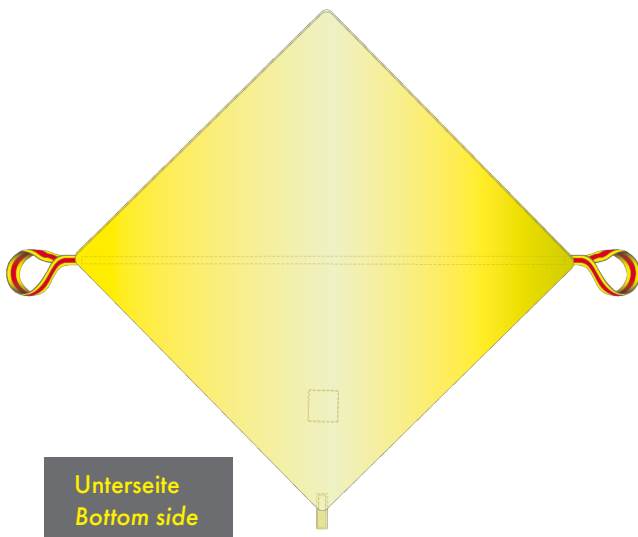
The warmth blanket shows an upper and bottom side which can be distinguished from each other by the position of the loop and hook tape straps. In order to roll up the warmth blanket again it is unfurled flat onto the floor. The diagonal full-length loop strap marks the upside. Now the lifting sling with carrying slings is hooked and firmly pressed onto it full-length.

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


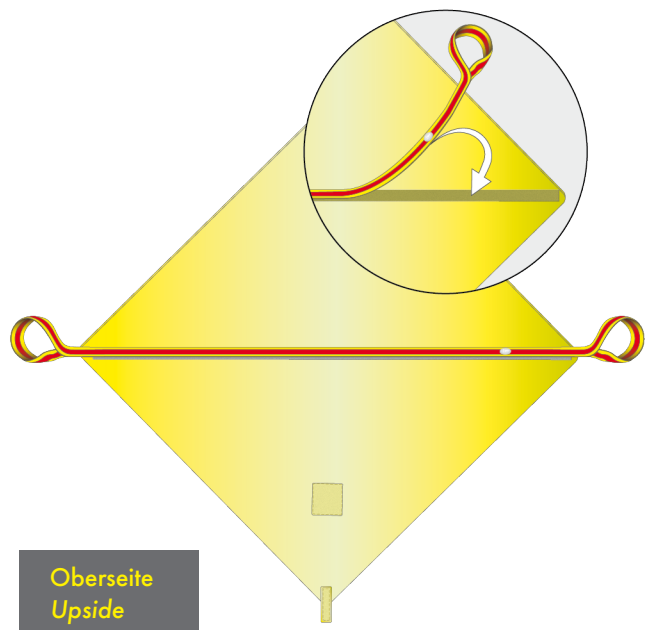
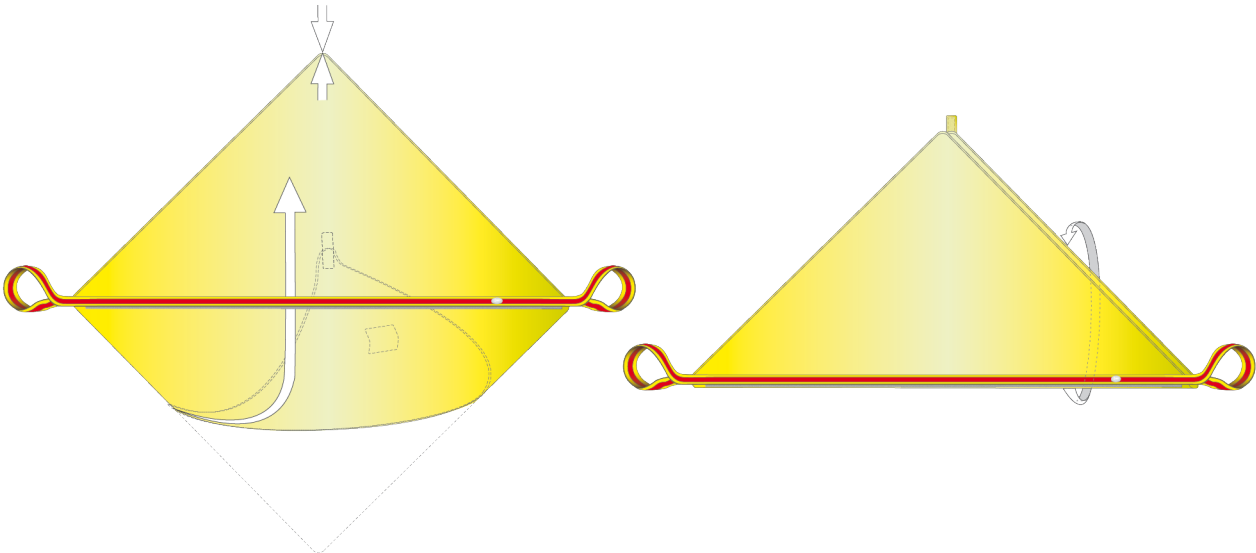
*Both free tips of the fleece blanket are laid on top of each other. The open tip of the blanket is fixed with hook tape in the end.*

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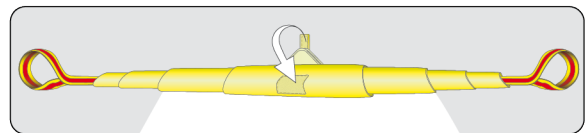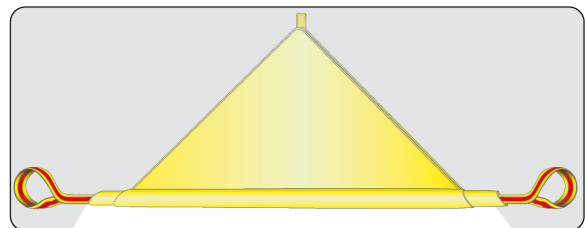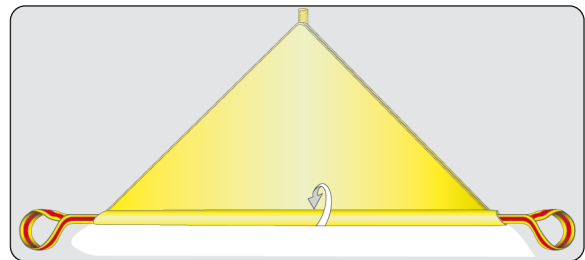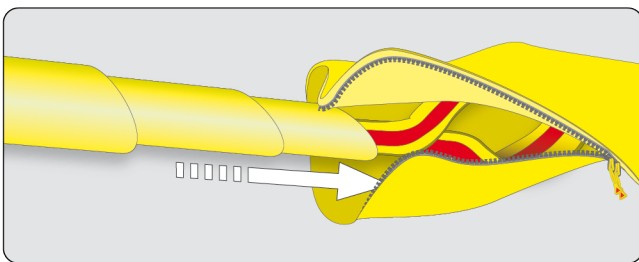*In order to put the rolled up warmth blanket with lifting sling back into the cover, the zip sliders are positioned in the centre of the cover. Now the rolled up warmth blanket is pulled through the remaining opening until the ends of the warmth blanket and the ends of the cover do match.*

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*After that the zip sliders are pulled to the ends. In doing so attention shall be paid to the warmth blanket not to move. The hand loops of the lifting sling protrude the same length from the ends of the cover. Thus the BOA is ready-for-use again. In order to put the PAX Rescue-BOA into the transport bag both ends are lead to the centre of the BOA and then both double strands are furled again. Folded in that way the BOA fits well into its transport bag.*

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## PAX Washing hints for the PAX Rescue-BOA:

As far as the process of washing is concerned the warmth blanket, cover and lifting sling are separated from each other. The lifting sling may not be washed but has to be cleaned by means of a wipe disinfection. To do this the indications of the disinfectant producer have to be observed.

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*The product has a lifetime of max. 5 years from purchase date.*

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*Every year of use the person in charge of medical products has to carry out a visual inspection of all seams, connections, handles and surface material and to record it at the end of these instructions for use. Defective products may not be used anymore and have to be disposed of.*

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 **Order here!**

 **PRAXISDIENST**  
Medical Supplies since 1953

Tel. / Phone: +49 (0)44 07 - 7 14 76 - 0  
Fax / Fax- : +49 (0)44 07 - 7 14 76 - 99  
Email: [info@pax-bags.de](mailto:info@pax-bags.de)

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The contents of these operating instructions are protected by copyright. Any use, including excerpts, requires written permission.
**X-CEN-TEK GmbH & Co. KG**