

Mach LED 300

DE	deutsch Gebrauchsanweisung	FI	suomi Käyttöohjeet	SE	svenska Bruksanvisning
EN	english User manual	HU	magyar Használati utasítás	SK	slovenčina Návod na použitie
FR	français Mode d'emploi	HR	hrvatski Uputa za uporabu	SL	slovenščina Navodila za uporabo
IT	italiano Istruzioni per l'uso	LT	lietuvių Naudojimo instrukcijos		
ES	español Manual de instrucciones	LV	latviešu Lietošanas instrukcija		
BG	български език Инструкция за употреба	NL	nederlands Gebruikershandleiding		
CS	Česky Návod k použití	NO	Norsk Bruksanvisning		
DA	dansk Brugsanvisning	PL	wersja polska Instrukcja obsługi		
EL	Ελληνικά Οδηγίες χρήσης	PT	português Manual de instruções		
ET	eesti Kasutusjuhend	RO	română Manual de utilizare		



MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug, SWITZERLAND



Congratulations on acquiring a new Mach LED 300 operating light.

Please read these Instructions for use very carefully.

1. Instructions for safe use

1.1 Intended user

The Mach LED 300 is a Class I medical product and may only be operated by trained medical staff.

1.2 Information and obligation of the user to check the product

Pay attention to the instructions for use when handling the lamp. These Instructions for use are part of the product and must therefore be stored in a place close to the product in order for the safety instructions and important information to be consulted at any time.

Make sure that the lamp is in satisfactory working order before every use. If there is obvious damage, unusual operating conditions, etc., the lamp must not be used.

1.3 Availability of the instructions for use

These instructions for use and a detailed handbook with a list of accessories and further tips on how to use the light can be found online at the following link:

<https://dr-mach.de/login/mach-led-300.html>



1.4 Intended use / contra-indications

The Mach LED 300 operating light is designed to illuminate an operating site in medical facilities (e.g. in a laboratory, in hospitals or doctor's practice) with focussed, low-glare, shadow-free light. It enables the user to perform a diagnosis or carry out medical interventions. The Mach LED 300 light is an operating light that is not fail-safe when used as a single light. It is not intended for use in areas where explosions are likely although it is permissible to use it in the vicinity of HF surgical equipment.

Permanent illumination of the open human eye should be avoided when illuminating the face area.

1.5 Technical data

Class of protection	I
IP protection class	IP 42
Input voltage (power supply 200W / 190W)	100-240 V AC, 50/60 Hz
Input voltage (light body)	24-30 V DC
Power consumption	32 W (Mach LED 300DF SC Spot) 46 W (Mach LED 300DF SC - camera preparation) 53 W (Mach LED 300DF SC - camera) 32 W (Mach LED 300MC - camera preparation) 39 W (Mach LED 300MC - camera)
Current	2.0 A max. (Mach LED 300DF SC Spot) 2.0 A max. (Mach LED 300DF SC - camera preparation) 2.3 A max. (Mach LED 300DF SC - camera) 1.3 A max. (Mach LED 300MC - camera preparation) 1.7 A max (Mach LED 300MC - camera)
Operating time	Continuous operation possible
Expected life ⁽¹⁾	10 years

¹ At the end of the expected (designed) service life, the lamp must be serviced more frequently for safe operation (see the manual for details).

1.6 Lighting technical data

	Mach LED 300DF SC	Mach LED 300DF SC Spot	Mach LED 300MC
Central light strength (distance 1 m)	160,000 lux	160,000 lux	160,000 lux
Light field diameter d10	189 mm	185 mm	184 mm
Light field diameter d50	100 mm	97 mm	104 mm
Residual light intensity (one shadower)	41 %	38 %	44 %

Residual light intensity (two shadowers)	42 %	47 %	44 %
Residual light intensity (normed tube)	100 %	100 %	100 %
Residual light intensity (normed tube, one shadower)	41 %	38 %	44 %
Residual light intensity (normed tube, two shadowers)	42 %	47 %	44 %
Illumination depth (20%)	965 mm	1520 mm	915 mm
Illumination depth (60%)	552 mm	628 mm	510 mm
Radiation intensity in the field (distance 1 m)	584 W/m ²	582 W/m ²	574 W/m ²
Max. radiation intensity in the field (distance 0.94 m)	588 W/m ²	-	-
Max. radiation intensity in the field (distance 0.90 m)	-	-	674 W/m ²
Max. radiation intensity in the field (distance 0.85 m)	-	602 W/m ²	-

For a complete overview of technical and lighting specifications, refer to the manual.

1.7 Installation/Maintenance/Repair

The lamp may only be installed, maintained or repaired by the manufacturer or by specially trained staff. Maintenance must be carried out at least every two years

1.8 Environmental conditions for operation

Ambient temperature: +5°C to +40°C
Relative air humidity: 30% to 75% RH
Air pressure: 700h Pat o 1060h Pa

1.9 Reporting obligations

Every serious incident which has occurred in connection with the product must be reported to the manufacturer and the competent authority.

2. Images on the device



This symbol indicates that the instructions for use must be followed.



Serial number of the product



Part number of the product



Address of the manufacturer



Date of manufacture and country of manufacture



EC conformity symbol



This symbol indicates that this is a medical device.



Reference to China RoHS / pollution control logo






















Positioning arrows



Instruction for disposal of the device

3. Safety instructions

	This symbol indicates possible sources of danger. Please also note the safety instructions and the hazard specification in the associated installation and operating instructions for the support arm system.
	Pay attention to the instructions for use when handling the lamp.
	Warning: to avoid the risk of an electric shock, this device must only be connected to a supply network that has a protective earthing conductor.
	A primary-side ON/OFF switch to isolate the system from the supply network must be provided on site. The switch must meet the requirements of IEC 61058-1 for nominal voltage peaks of 4 kV.
	This device is not designed for operation in environments enriched with oxygen.
	The lamp may only be used for the intended purpose. Otherwise, the manufacturer will not be liable for personal injury or damage to property.
	The lamp is equipped with a sterilizable handle at the factory and must only be used with this handle.
	Changes to the light are prohibited and will invalidate the manufacturer's certificate of conformity and all warranty claims.
	Use only the mains units (or transformers) approved or supplied by the manufacturer. Non-observance will void the conformity of the product and release the manufacturer from any claims under warranty.
	Installation, maintenance and repair work may only be carried out by the manufacturer or by specially trained staff.
	Maintenance must be carried out on the light at least every two years

	Additional equipment that is connected to medical electrical equipment, must conform to the relevant IEC- or ISO standards (e.g. IEC 60950 or IEC 62368 for data processing equipment). Moreover, all configurations must meet the requirements for medical electrical systems (see Section 16 of the latest version of IEC 60601-1). Anyone who connects additional equipment to medical electrical equipment, is configuring a medical system and is therefore responsible for ensuring that the system meets the requirements for medical electrical systems. In case of doubt, contact your local representative or our technical customer services.
	The simultaneous use of several lights to illuminate a wound area may result in the maximum allowed energy input being exceeded (1,000 W/m ²) and thus excessive heat development. It is the user's responsibility to ensure that the maximum allowed limit is not exceeded.
	The unprotected human eye can be damaged by direct light. Do not look directly into the light beam of the lamp. Do not point the light beam at the patient's unprotected eye continuously.
	Do not allow the laser beam to enter the eyes of the patient or user. The eyelid closing reflex of patients, in particular, may be impaired by this.
	When positioning the light body, there is a risk of injury (e.g. crushing) and collisions with other objects (inventory) or walls.
	Parts that fall off could injure the patient or lead to an infection of the wound area.
	Do not remove the rating plate or the warning labels.
	It is forbidden to carry out servicing or repair activities whilst the lamp is in use.
	Simultaneous touching of parts on the luminaire and the patient is not permitted.

4. Operating the Mach LED 300 light



ON/OFF switch of the light
(To turn off the lamp, press and hold the (ON/OFF) button for one second.)



Regulation of the brightness
Switching on the endoscopy mode



Regulation of electronic focus




Adjustment of the color temperature

5. Operating the camera remote control



Camera controls:

ON/OFF:
Switch between operation and stand-by

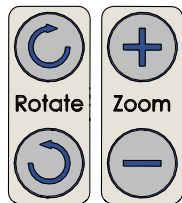

Rotate:
Rotate video image


Zoom:
Magnification of the video image



White balance:
adjusting the colour temperature to the actual environment



Still image

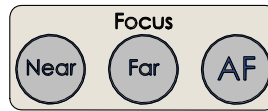

Focusing:
Autofocus – Switch on/off

Near/far – Adjust to near or far objects



Exposure
AE – Switching the automatic exposure control on/off.
Sun/Moon – Increase or decrease the brightness


6. Operating the Mach LED 300 manual remote control




Light's ON/OFF switch




Transfers the settings of light intensity, colour temperature, focus and endo-mode to other lights

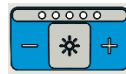

Adjustment of light intensity

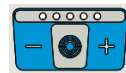


Adjustment of the electronic focussing

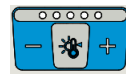

Adjustment of the colour temperature

7. Pairing

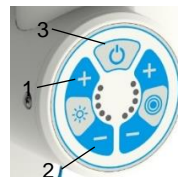

Option A: Pairing manual remote lamp control

Start pairing by pressing and holding (min. 3 sec.) of the key combination 1 (+), 2 (-) and 3 (ON/OFF)




Flashing LED 2 and LED 3 – pairing mode activated

Press any key of the remote control to transmit Sync address and WCOM address to the lamp

Note: No other hand-held remote control may be active or activated during pairing


Further information and overview of SYNC and WCOM addresses can be found in the manual for downloading - Link see Chapter 1.3

Option B: Pairing Camera

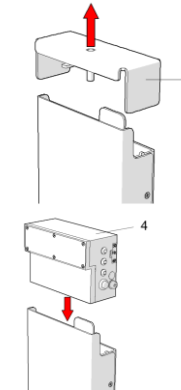

Radio receiver connection:
Pull the top cover 3 upwards

Insert receiver module 4 into the wall bracket

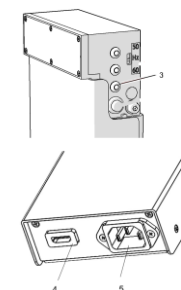


Radio receiver is now in configuration mode - green status LED 3 flashes continuously for approx. 1 minute
After one minute radio receiver is in normal operating mode - (green LED 3 flashes briefly)

Connect the monitor or other compatible device (recording device, splitter, etc.) with an HDMI cable to the HDMI connection 4 of the wall bracket.

Connect the power cable to connector 5

1. Switch on the camera with the remote control with the ON/OFF button in standby mode (permanently lit yellow LED in the camera)
2. Press the red button on the radio receiver and hold it pressed for at least 5 seconds (yellow LED flashes quickly)

3. Press freeze button of the remote control camera (facing the camera) five times in a row within 15 sec. - yellow LED of the camera flashes - pairing starts
4. Pairing takes a few minutes; after completion, the yellow LED in the camera and the blue LED on the radio receiver are lit continuously
5. Switch on the camera using the remote control - if there is a picture on the monitor - pairing successful; no picture on the monitor - repeat the procedure
Status of the radio link by displaying the blue LED 6:

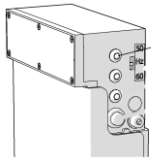

- LED lights up permanently: radio connection stable and permanent
- LED flashes quickly: Radio connection established, but video not displayed on monitor - check monitor cable
- LED flashes slowly: No radio connection established

8. Cleaning and disinfecting

Cleaning and disinfecting work must only be done by trained staff. The respective requirements must be observed for all cleaning and disinfection work (details can be found in the manual).

Housing/protective screen

The housing and the protective screen of the lamp body can be cleaned and disinfected with many common/commercially available materials. **Do not use cleaning agents or disinfectants containing active substances based on biguanides or phenols.** Cameras must be removed and the camera bay cover put on prior to cleaning and disinfection.

Furthermore, only cleaning agents approved for polycarbonate (PC) may be used to clean the protective screen. To protect against mechanical damage, always use a damp cloth (never a dry one) to wipe the protective screen and after cleaning, wipe with an anti-static agent (lint-free cloth).

Sterilisable handle

The handle must be cleaned/disinfected before each use. It can be steam sterilised (max. 200 sterilisation cycles for max. 5 minutes at a max. temperature of 134°C; details can be found in the manual).





Before installing the handle, check it for visible damage, fouling and the specified manufacturing date. Do not use damaged or dirty handles or handles that are more than two years old.

9. Faults

In the event of unusual operating conditions or a fault message displayed on the display, the lamp must not be used, as safe operation cannot be guaranteed. For troubleshooting, disconnect the lamp from the mains for about 30 seconds. In the event of continuous faults, a suitably trained service technician must be contacted, stating the error code.

10. Information on electromagnetic compatibility

Medical electrical equipment is subject to special precautionary measures with regard to electromagnetic compatibility (EMC). They may only be installed and put into service in accordance with the EMC instructions in the accompanying documents. The Mach LED 300 operating light has been tested for use in professional equipment of the health care system.

	The lamp is suitable for use with an RF surgical device. There must be a distance of at least 50 cm between the surgical light, including the suspension system, and the RF electrode cables.
	Portable and mobile RF communications equipment can affect medical electrical equipment and must not be used within 30 cm of the light, including the cable.
	The use of this equipment immediately next to other equipment or with other equipment in stacked form should be avoided since this may result in faulty operation. Should use in the aforementioned manner nevertheless be necessary, this device and the other equipment should be kept under observation to ensure that they are working properly.
	The use of different accessories, converters or cables to those that the manufacturer of this device has stipulated or made available may result in increased electromagnetic interference or reduced immunity to electromagnetic interference and to faulty operation.



Furthermore, the light must not be operated if the housing, cable or electromagnetic shielding is damaged.

Additional information on electromagnetic compatibility can be found in the manual.

11. Disposal



The light does not contain any harmful substances. The components of the light should be disposed of appropriately at the end of the product's life.

Take care that the material is carefully separated: The electrical circuit boards should be recycled appropriately. The housing of the light and the other components should be disposed of according to the materials they contain.

 **PRAXISDIENST**
Medical Supplies since 1953

 **Order here!**