TRI-BALL®

- (ES)** EJERCITADOR RESPIRATORIO
- (EN)** RESPIRATORY EXERCISER
- (FR)** EXERCISEUR RESPIRATOIRE
- (DE)** ATEMTRAINER
- (IT)** APPARECCHIO RESPIRATORIO
- (PT)** EXERCITANTE RESPIRATÓRIO
- (NL)** ADEMHALINGSTRAINER
- (CS)** TRENAŽÉR DÝCHÁNÍ
- (HR)** APARAT ZA VJEŽBE DISANJA
- (HU)** LÉGZÉSTRÉNER
- (PL)** URZĄDZENIE DO ĆWICZEŃ ODDECHOWYCH
- (SL)** VADBENI RESPIRATOR
- (LT)** KVĖPAVIMO TRENIRUOKLIS
- (AR)** ممرن الجهاز التنفسي
- (MS)** ALAT SENAMAN PERNAFASAN


Leventon


A Werfen Company

LEVENTON, S.A.U.
Newton, 18-24
08635 Sant Esteve Sesrovires
Barcelona, Spain

Componentes / Components / Composants / Komponenten / Componentes / Componenti /
 Componenten / Součásti / Dijelovi / Alkatrészek / Elementy / Sestavni deli / Komponentai /
 المكونات / Komponen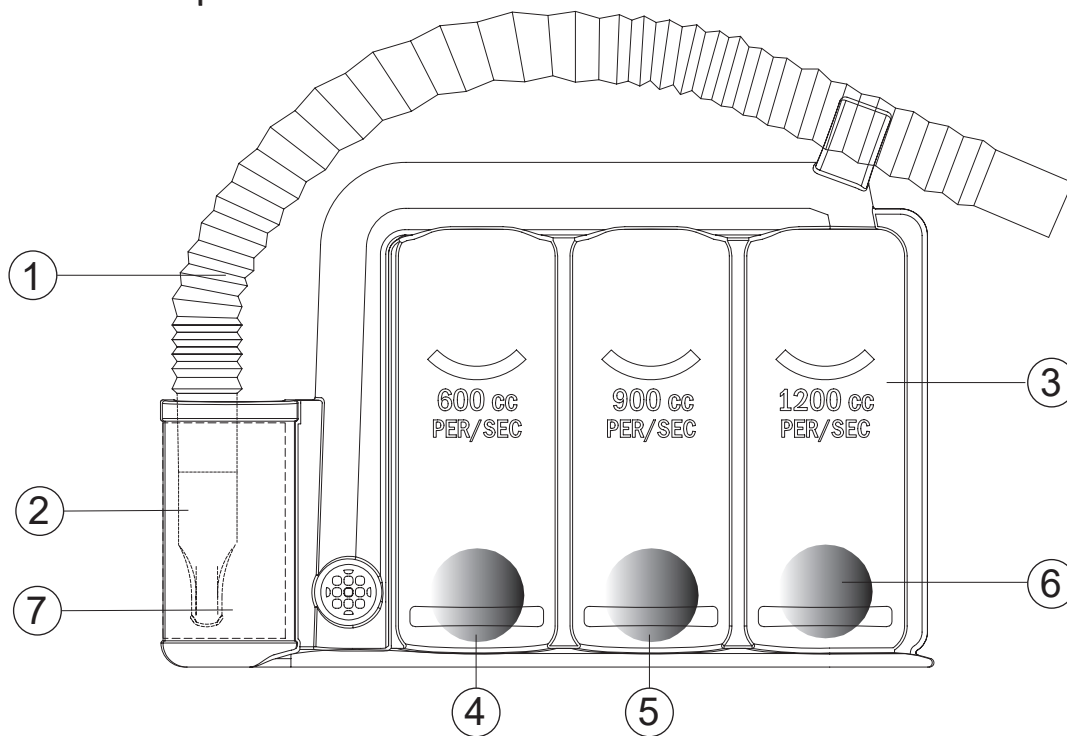

Item	Nombre
①	Tubo de aspiración / Inhalation tube/ Tube d'inhalation / Inhalationsschlauch / Tubo per inalazione / Tubo de inalação / Inhalatiebuis / Inhalační trubice / Cijev za udisanje / Belélegző cső / Rurka do inhalacji / Cev za vdihavanje / [kvépimo vamzdelis / أنبوب الاستنشاق / Tiub penyedutan
②	Boquilla / Mouthpiece/ Embout buccal / Mundstück / Boccaglio / Bocal / Mondstuk / Náustek / Usnik / Szájrészt / Ustnik / Ustnik / Kandiklis / القطعة المخصصة للضم / Pelekap mulut
③	Carcasa / Casing / Boîtier / Gehäuse / Contenitore / Bolsa / Behuizing / Pouzdro / Kućište / Ház / Obudowa / Ohišje / Korpusas / الأنبوب اللولبي المتصل / Selongsong
④	Bola 1 / Ball 1 / Balle 1 / Kugel 1 / Sfera 1 / Bola 1 / Bal 1 / Kulička 1 / Kuglica 1 / 1. golyó / Kulka 1 / Kroglica 1 / 1 rutuliukas / الكرة 1 / Bola 1
⑤	Bola 2 / Ball 2 / Balle 2 / Kugel 2 / Sfera 2 / Bola 2 / Bal 2 / Kulička 2 / Kuglica 2 / 2. golyó / Kulka 2 / Kroglica 2 / 2 rutuliukas / الكرة 2 / Bola 2
⑥	Bola 3 / Ball 3 / Balle 3 / Kugel 3 / Sfera 3 / Bola 3 / Bal 3 / Kulička 3 / Kuglica 3 / 3. golyó / Kulka 3 / Kroglica 3 / 3 rutuliukas / الكرة 3 / Bola 3
⑦	Etiqueta paciente / Patient label / Étiquette du patient / Patientenspezifisches Etikett / Etichetta paziente / Rótulo do doente / Patiënt-etiket / Štítek pacienta / Oznaka za pacijenta / Betegcímke / Etykieta dla pacjenta / Nalepka za bolnika / Paciento etiketè / ملصق المريض / Label pesakit

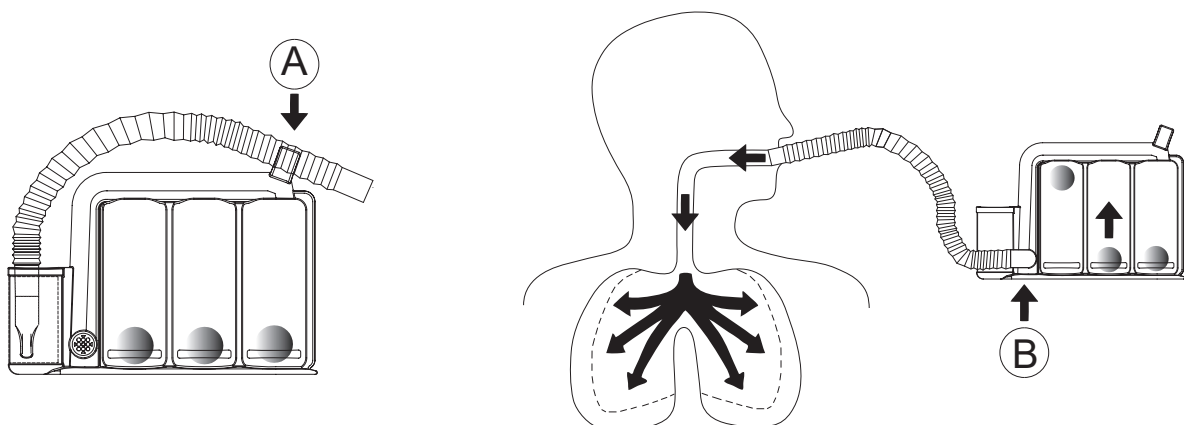DESCRIPTION.

The TRI-BALL® respiratory exerciser is a medical device that comes into contact with the mucous membrane of the body surface area (mouth). The system consists of a mouthpiece, an inhalation tube and a casing with three independent chambers and their respective inspiration flows, with a different coloured ball in each chamber. The casing also features a holder in which to place the mouthpiece.

INDICATIONS FOR USE.

The TRI-BALL® respiratory exerciser is designed to assist patients who have undergone abdominal or thoracic intervention to regain normal breathing. Deep breathing exercises play an important role in the recovery of these patients. The device helps patients to regain their normal breathing patterns sooner and reduces the risk of respiratory complications. The product is indicated for both adult and paediatric patients of over 8 years of age.

CONTRAINDICATIONS.

The TRI-BALL® respiratory exerciser is not indicated for:

- Patients who cannot be trained to use the device correctly or monitored.
- Patients with a physical disability or patients who are unable to understand or demonstrate the correct use of the device.
- Patients who cannot breathe deeply and effectively due to pain, diaphragm dysfunction or opioid analgesia.
- Patients with respiratory muscular fatigue who are unable to generate an adequate inspiration with a vital capacity of <10 ml/kg or an inspiratory capacity of <33% of normal predicted.

INSTRUCTIONS FOR USE.**Warnings:**

A1. Do not use this device in the event the product is damaged or any of its parts are broken, as this may result in the product malfunctioning in the worst case scenario.

A2. This use of this product is restricted to a single patient.

A3. The product is valid for 30 days or 1,000 cycles of use.

Assembling the product:

1. Remove TRI-BALL® from its bag and the tube from its case (A).
2. Connect the open end of tube to the hole (B).

Instructions for use:

1. Prior to beginning your exercise, breathe out normally and then place the mouthpiece in your mouth, wrapping your lips tightly around it.
2. To achieve a low inspiratory flow rate, inhale until the first ball rises. Maintain this position for as long as possible. The second and third balls should remain in their initial position.
3. To obtain a medium inspiratory flow rate, inhale until the first and second balls have risen to the top of their chambers. The third ball should remain in its initial position.
4. To obtain a maximum inspiratory flow rate, inhale deeply until all three balls have risen to the top of their chambers.
5. Release the mouthpiece and exhale.
6. Rest for a moment breathing normally.
7. Repeat the exercise as instructed by the physician.

Precautions:

P1. Wash the mouthpiece prior to use.

P2. The patient should place the TRI-BALL® device in an upright position at face level whenever using the respiratory exerciser to ensure it functions correctly.

P3. If the respiratory exerciser is dropped on the floor accidentally or intentionally, do not reuse the product as it may be broken, which could result in the product malfunctioning in the worst case scenario.

P4. If you have any questions on how the device works or if you are feeling unwell, talk to your doctor or therapist.

CLEANING AND STORAGE INSTRUCTIONS.

1. Before and after using the TRI-BALL® device, wash the mouthpiece with warm water, place it in the mouthpiece holder to dry and put TRI-BALL® back in its original bag until further use.

Precautions:

P1. Do not store in moist environments.

SAFE DISPOSAL OF THE PRODUCT.

1. Dispose of the product in accordance with the healthcare facility's protocol or healthcare personnel instructions. Any serious product-related incident should be reported to the manufacturer and the competent authority.

CLARIFICATION ON SYMBOLS.

The symbols used on the TRI-BALL® label comply with ISO15223-1: 2016. The following additional symbols have been used:

The product is a Medical Device.

The product may be used several times by the same patient.

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