
PHILIPS


RESPIRONICS

InnoSpire Deluxe

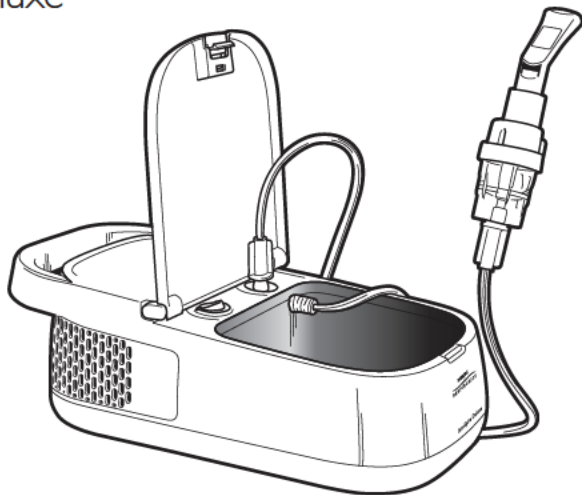InnoSpire Deluxe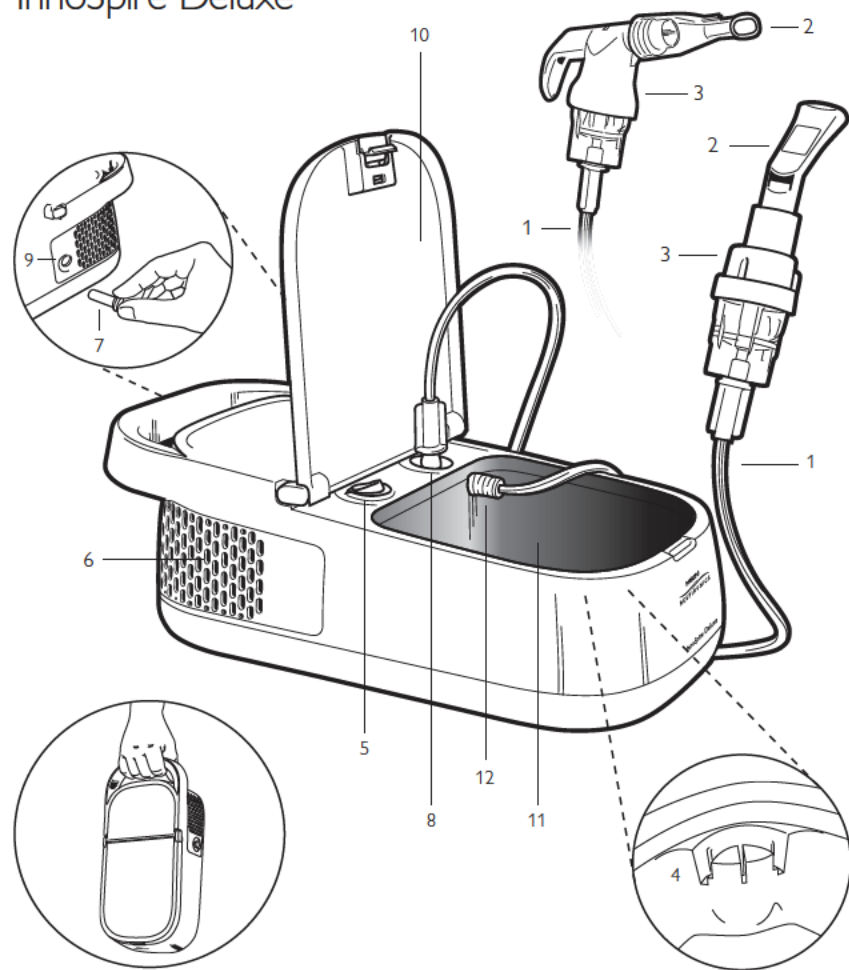

Index

English	EN	4
Български	BG	10
Česky	CZ	16
Hrvatski	HR	22
Dansk	DA	28
Deutsch	DE	34
Español	ES	40
Eesti	EE	46
Français	FR	52
Ελληνικά	EL	58
Indonesia	ID	64
Italiano	IT	70
Latviski	LV	76
Lietuviškai	LT	82
Magyar	HU	88
Nederlands	NL	94
Norsk	NO	100
Polski	PL	106
Português	PT	112
Română	RO	118
Srpski	SR	124
Slovenčina	SK	130
Slovenščina	SL	136
Suomi	FI	142
Svenska	SV	148
ภาษาไทย	TH	154
Tiếng Việt	VI	160
Türkçe	TR	166
العربية	AR	172
한국어	KO	178
עברית	IL	184
EMC data sheets (EN)		190

- 1. Nebulizer tubing
- 2. Nebulizer mouthpiece
- 3. Nebulizer handset
- 4. Nebulizer holder (inside view)
- 5. On/Off power switch
- 6. Compressor air openings
- 7. Air filter (side view)
- 8. Compressor air outlet
- 9. Air inlet for filter (side view)
- 10. Lid
- 11. Storage compartment
- 12. Power cord location

Instructions for use

InnoSpire Deluxe

Important: Read these instructions carefully before using this device. If you have any questions or problems with the product, please contact your healthcare provider or Philips Respironics Customer Service at +1 724 387 4000.

General information

Intended use: This product is intended to provide a source of compressed air for medical purposes. It is to be used with a pneumatic (jet) nebulizer to produce aerosol particles of medication for respiratory therapy for both children and adults.

WARNINGS

- 1 Use the device for its intended purpose as described in this instruction manual as a compressor nebulizer system for aerosol therapy. Follow your doctor's instructions. Any other use constitutes improper use and may be dangerous. The manufacturer cannot be held liable for any damage caused by improper, incorrect or unreasonable use or if the unit is connected to electrical installations not complying with safety regulations.
 - It is recommended to have a backup device (e.g. MDI or battery operated compressor) for respiratory delivery in case a situation arises when your nebulizer cannot be used (e.g. during a power outage or if your compressor becomes inoperable for any reason).
 - If the device has been outside of the operating conditions, allow to come back to normal operating specifications before use.
 - Any liquid spilled on or inside the unit should be allowed to dry before operating.
- 2 Keep this manual for future reference.
- 3 Use the product in a clean environment; one that is free from dirt, dust, pet hair, etc.
- 4 Device should not be used around flammable substances, or in the presence of a flammable anesthetic mixture with air, oxygen or nitrous oxide.
- 5 No modification of this equipment is allowed.
- 6 It is normal for the device to be warm to the touch when in use.

- 7 Do not stack the compressor with other equipment.
- 8 **Electromagnetic information:** Portable and RF communication devices such as cellular phones, pagers, etc. can interrupt operation of electrical medical equipment. For this reason, your compressor must be placed far enough away from these devices to prevent interference. This device meets IEC60601-1-2 standard for electromagnetic compatibility (EMC). EMC data sheets are available at the back of this document. EMC data sheets in other languages are available from Philips Respironics customer service at +1 724 387 4000. The CE mark on the product denotes compliance with all applicable EU Directives. Note that the Notified Body number does not apply to the RoHS (restriction of the use of certain hazardous substances in electrical and electronic equipment) Directive.
- 9 In case of failure and/or malfunction, read the "Troubleshooting" section. Do not open the compressor housing as this is not a serviceable product.
- 10 The proper function of the device can be compromised if original replacement parts (e.g., filters) are not used.
- 11 To reduce risk of electric shock, DO NOT DISASSEMBLE.
- 12 Indoor use only.
- 13 Do not use the device without the filter.
- 14 When using any electrical appliance, certain important safety measures must always be followed, including the following:
 - Use only manufactured-supplied original accessories and components;
 - Never submerge the unit in water as the device is not protected against water penetration;
 - Never touch the unit when your hands are wet or moist;
 - Do not leave the unit outdoors;
 - Place on a stable and horizontal surface when operating;
 - Make sure the air vent openings are not obstructed;
 - Do not allow unsupervised children or infirm individuals to use the unit;
 - Do not turn off the unit by simply pulling out the plug from the electrical outlet;
 - Do not use in a ventilator circuit;
 - Do not use if the device has cracks or is damaged.
- 15 Make sure that the electrical rating shown on the rating label on the bottom of the unit corresponds to your main voltage and frequency before plugging in the device.
- 16 Do not use any adapter; single or multiple, and/or extension cord.
- 17 Do not leave the unit plugged in when not in use. Unplug the device from the electrical outlet when not in operation.
- 18 Follow the manufacturer's instructions for installing the device. The manufacturer cannot be held responsible for any damage caused by incorrect installation.

- 19 The power supply cord cannot be replaced by the user. In case the power supply cord becomes damaged, contact Philips Respironics customer service for replacement (there is no service option).
- 20 Do not over fill the storage compartment with accessories.
- 21 Before performing any cleaning operation or filter replacement, disconnect the device from the main supply by unplugging it.
- 22 Small parts can be inhaled, swallowed or dangerous. Cable, due to its length, may result in provoking strangulation or asphyxiation. Do not leave the device alone with a small child or physically challenged individual or an individual with learning difficulties.
- 23 If you no longer use the device, dispose of according to the local regulations in force.
- 24 Remember to:
 - Use this device only with medications prescribed by your doctor;
 - Use the accessories recommended by your doctor.

Instructions for use

Remove the compressor from carrying case if supplied before use.

- 1 After unpacking the device, check the compressor and accessories for visible damage or defects such as cracks in the plastic housing which could expose electrical components. Contact Philips Respironics customer service or your product distributor to report any damaged product.
- 2 When taking a treatment, sit upright and relaxed.
- 3 Plug the compressor into the electrical outlet.
- 4 Refer to the nebulizer and accessory instructions for use before treatment.
- 5 To start the treatment, switch the device ON by pressing the ON / OFF switch onto the "I" position.
- 6 When the treatment has been completed, switch the unit OFF by setting the switch position to "O" and disconnect the plug from the electrical outlet.
- 7 Do not close the lid when the nebulizer tubing is attached.

Cleaning and maintenance

During cleaning, make sure the power cord is not plugged in the electrical outlet.

Cleaning of the compressor, nebulizer and accessories

Use a damp cloth to wipe down the outside of the compressor at least once each month. Refer to nebulizer and accessory instructions for use for cleaning of the accessories and nebulizer.

Replacement of the nebulizer

The nebulizer and other accessories should be replaced according to its instructions.

Replacement of the filter

The filter is to be replaced when it appears discolored or wet.

Troubleshooting

The device does not turn on

- Make sure the plug is firmly fitted to the electrical outlet.
- The thermal protector may have been activated by the following reasons:
 - a) the device was used near a heater or in a hot environment
 - b) the air vent openings (6) are obstructed and do not allow for the motor ventilation
 If this is the case set the ON/OFF switch to 'O' and let it sit for at least 60 minutes. In the event of malfunction (insufficient pressure and flow) turn the unit off for 10 seconds and turn back on.

After each cleaning, check that the unit is working properly by verifying that air still comes out of the nozzle when the unit is turned to the "ON" position.

Unauthorized repair voids warranty.

If the device does not work properly after attempting these troubleshooting steps, contact your authorized product distributor or Philips Respironics Customer Service at +1 724 387 4000.

Maintenance and repairs

Never open the device. There are no serviceable parts within the unit. The compressor requires no lubrication or maintenance.

ENGLISH

Technical specifications

Electricity supply	230V/50Hz 1 Amp
Thermal overload protector	Resettable thermal fuse; functioning temperature 150 C°
Compressor max. pressure	360 kPa (3.6 bar)
Average flow rate	7 LPM 70 kPa (0.7 bar)
Max flow rate	9.8 LPM
Weight	2.2kg
Size	380mm x 170mm 110mm
Noise level	58±3dBA

Technical specifications

Electricity supply	220V/60Hz 1.2 Amp
Thermal overload protector	Resettable thermal fuse; functioning temperature 130 C°
Compressor max. pressure	268 kPa (2.68 bar)
Average flow rate	6 LPM 70 kPa (0.7 bar)
Max flow rate	10.7 LPM
Weight	2.0kg
Size	380mm x 170mm x 110mm
Noise level	58±3dBA

Class II device, Type BF device (device with specific protection against electrical hazards).

DOUBLE INSULATED

Reference to standards

Electric safety standards IEC 60601-1
Electromagnetic compatibility according to IEC 60601-1-2

Ambient conditions

Storage conditions	
Temperature	MIN -25 °C – MAX 70 °C (-13 °F – 158 °F)
Humidity	MIN 10% RH – MAX 95% RH
Operating conditions	
Temperature	MIN 5 °C – MAX 40 °C (41 °F –104 °F)
Humidity	MIN 10% RH – MAX 95% RH
Operating atmospheric pressure	Performance may vary depending on altitude above sea level, barometric pressure and temperature

Symbol glossary

ON (power)	ATTENTION	Class II, double insulated
OFF (power)	Alternating current	Type BF applied parts
Serial number	Separate collection	Indoor use only
Follow instructions for use	Manufacturer	Temperature limitation
Atmospheric pressure	Humidity	

IP21 Housing Ingress Protection rate: IP21 (protected from the penetration of solid bodies with dimensions greater than 12 mm. Protected from the penetration of vertically falling water drops).

Warranty

Respironics, Inc warrants the compressor to be free from defects in materials and workmanship under normal use and operation for a period of 3 years from date of purchase from Respironics, Inc. The warranty is limited to repair or replacement, at Respironics, Inc's sole option, of any such component or equipment claimed to be defective when claim is shown to be bona fide by evaluation by Respironics, Inc. This warranty does not extend to any components or equipment subjected to misuse, improper operation, accidental damage, or unauthorized repairs, and is not extended to charges of, or for, labor repairs. All items returned must be properly packaged and shipped, prepaid, by the product distributor servicing the unit. Respironics, Inc shall not be liable to purchaser or others for loss of use of equipment or for indirect, or incidental or consequential damages that might arise.

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A. Instructions for use

- Innospire Deluxe needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in this document;
- Portable and mobile RF communications equipment can affect Innospire Deluxe.

B. Technical description

- Warning** that the use of **accessories, transducers and cables** other than those specified with the exception of transducers and cables sold by the manufacturer of the Innospire Deluxe as replacement parts for internal components, may result in increased EMISSIONS or decreased IMMUNITY of the Innospire Deluxe.
- Warning** that the Innospire Deluxe should not be used adjacent to or stacked with other equipment.

Guidance and manufacturer's declaration – electromagnetic emissions		
The Innospire Deluxe is intended for use in the electromagnetic environment specified below. The customer or the user of the Innospire Deluxe should assure that it is used in such an environment.		
Emissions	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The Innospire Deluxe uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Innospire Deluxe is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A (not applicable for 120v rated devices)	
Voltage fluctuations/flicker emissions IEC61000 3-3	Complies (applicable to 50Hz rated devices only)	

4.

Guidance and manufacturer's declaration – electromagnetic immunity			
The Innospire Deluxe is intended for use in the electromagnetic environment specified below. The customer or the user of the Innospire Deluxe should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines and dc lines	±2 kV for power supply lines and dc lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV line(s) and neutral ±2 kV line(s) to earth	±1 kV line(s) and neutral ±2 kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC61000-4-11	<5 % U _T (>95 % dip in U _T) for 0,5 cycle 40 % U _T (60 % dip in U _T) for 5 cycles 70 % U _T (30% dip in U _T) for 25 cycles <5 % U _T (>95 % dip in U _T) for 5s	<5 % U _T (>95 % dip in U _T) for 0,5 cycle 40 % U _T (60 % dip in U _T) for 5 cycles 70 % U _T (30% dip in U _T) for 25 cycles <5 % U _T (>95 % dip in U _T) for 5s	Mains power quality should be that of a typical commercial or hospital environment. If a dips or an interruption of mains power occurs, the current of the Innospire Deluxe may be dropped off from normal level, it may be necessary to use uninterruptible power supply or a battery.
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE U _T is the a.c. mains voltage prior to application of the test level.			

5.

Guidance and manufacturer's declaration – electromagnetic immunity			
The Innospire Deluxe is intended for use in the electromagnetic environment specified below. The customer or the user of the Innospire Deluxe should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3Vrms 150kHz to 80MHz	3Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the Innospire Deluxe, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1.2\sqrt{P}$ $d = 1.2\sqrt{P}$ 80MHz to 800MHz $d = 2.3\sqrt{P}$ 800MHz to 2,5GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a , should be less than the compliance level in each frequency range ^b . Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	3V/m 80MHz to 2.5GHz	3V/m	
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Innospire Deluxe is used exceeds the applicable RF compliance level above, the Innospire Deluxe should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Innospire Deluxe. b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.			

6.

Recommended separation distances between portable and mobile RF communications equipment and the Innospire Deluxe			
The Innospire Deluxe is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Innospire Deluxe can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Innospire Deluxe as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1.2\sqrt{P}$	80 MHz to 800 MHz $d = 1.2\sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0,1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter; where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.			
NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

InnoSpire Deluxe is a trademark of Koninklijke Philips N.V.
and its affiliates, all rights reserved.

© 2014 Koninklijke Philips N.V. All rights are reserved.

CE0123

Manufactured for
Respironics Respiratory Drug Delivery (UK) Ltd
Chichester Business Park, City Fields Way
Tangmere, Chichester, PO20 2FT UK

Made in China

www.philips.com/respironics

PN 1113054 R01